


Decorative Safety Assist Bar

Installation Manual

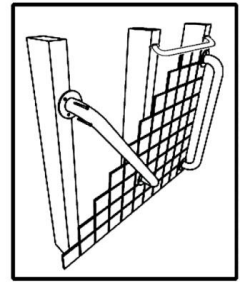

EASY INSTALLATION


According to different styles of Decorative Safety Assist Bar you purchased, locate an appropriate position in bathroom with sufficient space on wall. The safety assist bar can be installed horizontally, vertically, or at an angle based on your need.

- 1) Use a stud sensor to locate the wall studs.
- 2) Hold the safety assist bar at the desired location on the wall. Mark the position of each mounting bracket of the safety assist bar on the wall, ensuring stud location to secure mounting. Use heavy duty wall toggles/molly when drilling into substrate with no studs. (see installation recommendations below)



(Above style for reference only)

- 3) Detach the mounting plates inside the brackets by using Hex Key (provided) to unscrew the locking pins at the pin-hole on the side of each bracket. Hold the mounting plate on the marked position of each bracket on wall, and then mark the screw hole locations on wall through the mounting plates.
- 4) Take away the mounting plates. Drill holes at the marked screw hole locations. Use a special tipped drill bit if drilling through tile/marble.
- 5) Install all the mounting plates first to the wall with the stainless steel screws provided. Then mount the safety assist bar to the secured mounting plates on wall by screwing back the locking pins through the pin-hole on the side of each bracket with the Hex Key.

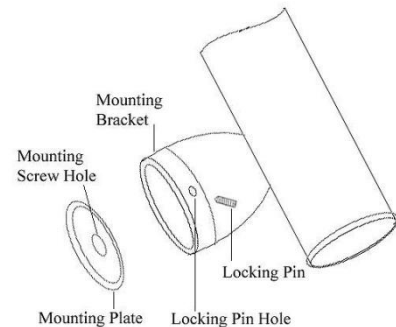

Installing into Wall with Wood Studs :

Position safety assist bar on wall with mounting plates desirably at wood studs location. Drill 5/32" holes into studs or backing of wall for each end of safety assist bar according to the marked screw hole position of each mounting plate. Use stud sensor to locate stud. Install mounting plates with #10 stainless steel screws (provided).

Important : Mounting to a secure stud ensures weight limit to be in compliance with ADA and Safety recommendations.

* When wood studs are not available at the desired location, a grab bar fastening system is recommended to meet ADA and Safety requirements. (not included, available at most hardware / home centers)

Installing into Dry Wall (no studs) :

Drill 1/2” holes according to the marked screw holes position of each mounting plate. Then, install mounting plates with 3/16” x 3” toggle bolts (not included) for each mounting plate, or special grab bar fastening system.

*When wood studs are not available, a grab bar fastening system is recommended to meet ADA and Safety requirements. (not included, available at most hardware / home centers)

Installing into Tile Wall :

Use a stud sensor to locate the wall studs. On ceramic tiles, try to locate screw hole(s) on grout line on a marked stud. If mounting on tile surface, use masking tape to mark & to prevent sliding during drilling. Drill hole with 1/4” glass drill bit according to the marked screw holes position of each mounting plate. Install #10 plastic anchors, and mount with #10 stainless steel screws (provided) for each mounting plate.

*When wood studs are not available, a grab bar fastening system is recommended to meet ADA and Safety requirements. (not included, available at most hardware / home centers)

Important

: To check if safety assist bar is securely mounted after assembly, pull the safety assist bar outward and push downward. If loose or not mounted properly, unscrew the safety assist bar and re-adjust the mounting location with better support on wall.

: Safety assist bar should only be installed on area of wall structurally sound as suggested above. If you are not sure, consult with a qualified contractor. Do not install on shower enclosure with insufficient construction strength.

WT International Inc.
4100 Cedar Commercial Drive NE,
Cedar Springs, MI 49319, U.S.A.
1-888-635-0007

sales@wtimfg.com

ARISTA[®] is a registered trademark of WT International Inc.